***drive***

 **DeVilbiss<sup>®</sup>**  
HEALTHCARE

**Bath Stool with Rotating Seat**

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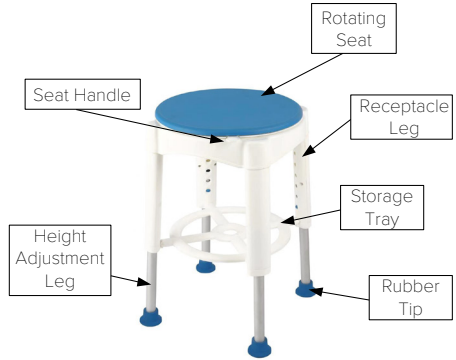
## Assembly Instructions

1. Remove contents from carton.
2. Assemble legs by inserting the height adjustment legs into the rubber tip. Make sure to put the tips on the end without the brass push pin. Repeat for 3 other legs.

3. Insert the height adjustment leg into larger leg receptacle by depressing the brass push pin. Repeat for 3 other legs.
4. Lay seat upside-down on a flat surface. Insert each leg into the seat depressing the brass push pin on the leg receptacle and inserting into the bottom of the seat. A click can be heard when the leg is securely in place. Repeat for 3 other legs.
5. Turn the bath bench right-side-up. Attach the soap tray by placing it between the legs and inserting the cross braces into the receptacles on the white leg collars.

## Adjusting Leg Height

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. To be sure legs are locked into place, push down on or pull up on the legs.

## Rotating Seat

1. To rotate seat, pull up on handle until seat is free of one of the 4 locked positions and rotate to desired position. Seat can rotate in both directions.

Note: Built into the seat frame are 4 locking positions which will secure the seat in place to allow the user to safely stand up from or sit down on the seat. Make sure the seat is in one of the locked positions before standing or sitting.

## Replacing Seat

The rotating seat and the two ball bearing rings that move the seat can be replaced if necessary. To remove seat and ball bearing rings, turn the lock knob under the seat clockwise until removed. Remove seat and ball bearing rings. Replace seat, ball bearing rings and lock knob. Tighten until secure.

## Additional Precautions

1. Inspect all parts for shipping damage before assembly. If there is shipping damage – DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
4. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
5. DO NOT stand on chair.
6. Only to be used on a flat and level surface.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

## Lifetime Limited Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorised Drive dealer.

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**Drive DeVilbiss Healthcare Ltd**
**Drive DeVilbiss Sidhil Ltd**