TECHNICAL DATA SHEET

*Sterile 0.9% solution of sodium chloride, in unidose
containers*

LABORATOIRES GILBERT

**SOLUTION DE CHLORURE DE SODIUM
STERILE 0,9%, EN RECIPIENT UNIDOSE**

*STERILE 0.9% SOLUTION OF SODIUM
CHLORIDE, IN UNIDOSE CONTAINERS*

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Sterile 0.9 % solution of sodium chloride is a sterile non injectable physiological saline solution, preservative-free. Presented in single-dose vials, sterile 0.9 % solution of sodium chloride is hygienic, easy and safe to use.

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Ocular hygiene: Medical device class IIb under Rule 21 - Point 4.

Nasal hygiene: Medical device class IIa under Rule 21 - Point 3.

Auricular hygiene: Medical device class IIa under Rule 5 - Point 3.

Wound cleansing: Medical device class IIb under Rule 4 - Point 3.

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- *Ocular hygiene: as an eyewash or bath to cleanse the eyes.*
- *Nasal hygiene: to irrigate or cleanse dry or stuffed up noses.*
- *Auricular hygiene: flush out solubilised earwax deposits and rinse the external auditory canal after using a surfactant auricular solution.*
- *Wound cleansing*

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- *Nasal and ocular hygiene: may be used daily as a nasal and eye wash 1 to 6 times a day, as needed.*
- *Auricular hygiene and wound cleansing: not applicable.*

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Instruction for use

The single-dose container is disposable and for individual use. Detach a vial and open it by twisting off the top.

- For nasal hygiene

Tilt the head back. Delicately insert the tip into one nostril and gently squeeze the vial. Repeat the operation in the other nostril. Lift the head up to allow the mucous to run out and then wipe away the excess. Do not blow your nose violently for 10 minutes after administering.

- For ocular hygiene

Pour a few drops of solution into each eye, taking care not to touch the surface of the eye with the vial. Wipe away excess liquid with sterile gauze.

- For ear hygiene

Rinse the external auditory canal with sterile 0.9 % solution of sodium chloride. Use 1 vial of sterile 0.9 % solution of sodium chloride for both ears. Wipe away the surplus liquid with a gauze pad.

- For cleansing wounds

Rinse the wound, using the entire contents of one vial and taking care not to touch the wound with the vial.

Precautions for use

Do not use the same single-dose for the different uses.

- For nasal hygiene

For babies, squeeze drops with a minimum amount of pressure to avoid any risk of contamination of the middle ear.

- For ocular hygiene

Wait 15 minutes before using any other eye treatment product.

- For ear hygiene

Ear use of this medical device in case of perforated eardrums, history of ear surgery or chronic ear conditions is not recommended or should be the subject of prior advice from your doctor or your pharmacist.

- For cleansing wounds

Wipe away the excess liquid from the unaffected area only.

MANIFESTATIONS INDESIRABLES / SIDE EFFECTS

Non attendues.

Not expected.

CONTRE-INDICATIONS / CONTRAINDICATIONS

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The single-dose container is disposable and for individual use. Do not inject. Do not swallow.

Do not use a single-dose vial that is open or damaged due to the risk of rapid bacterial contamination.

If symptoms persist or worsen, consult your doctor.

Report all serious incidents that occur in link with this device to the Laboratoires Gilbert – website : www.labogilbert.fr and/or to the national competent authority.

Keep out of the reach and sight of children.

CONDITIONS DE CONSERVATION / STORAGE CONDITIONS

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Do not use after the expiry date stated on the single-dose vial.

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