
Marks	B x F	C	D x G	E	H	Top thickness		Max
Easy	40x60-85 cm	0 to 90°	40x60 cm	70 to 105 cm	10.5 cm	1.8 cm	7 kg	10 kg

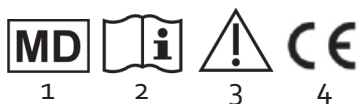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**EN** - Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

### 1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, castors...).
- ♥ Check regularly if all parts are tightened.
- ♥ Use only in proper way : don't stand on the table.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ After assembling the table, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ In case you didn't use your table for a long time, let it checked by the technical staff, before use.
- ♥ The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

### 2. Intended use, indications and contraindications

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. Each step is described in **this user's manual**.

In case of doubt, don't hesitate to ask again.

#### Intended use:

This bed table is a functional aid for nursing which makes eating, drinking and reading possible while sitting comfortably in bed.

#### Indications:

This product aims at assisting the medical staff when a nursing bed or a bed frame is used for a bedridden patient; e.g. in case of inability for the patient to eat independently out of their bed. The table surface can also be used by the patient for storage.

#### Contraindications:

Dementia; Upper limbs fractures or severe muscle weaknesses resulting in the patient's inability to raise the upper limbs.

#### Mounting:

To mount the table follow the below steps which are corresponding to the above diagrams.

**Step 1:** Check for damage. If you see any, or suspect a fault, do not assemble the product. Check content as described above.

**Step 2:** Slide part (2) into part (1), place the washer (4) and screw loosely using the lock nut (3). The new part obtained is numbered (6).

**Step 3:** Take the plate (7), put it flat on the ground. Slide part (6) into the plate and tighten with the lock nut (3). The new part obtained is numbered (8).

**Step 4:** secure the tube with castors (10) on the frame (11) using the screws (5); screw the lock nut (9) on the frame.

**Step 5:** Insert the upper part (11) into the base frame.

**Beware : Ensure that the table is assembled as described, and that all components are securely attached, before use. Check stability before use.**

#### Operation instructions :

This overbed table can be used to eat, drink etc. while sitting in bed or in a chair. It is adjustable both in height and width and equipped with 4 castors (2 of which can be locked).

**Beware : Do not lean on the suspended end of the table top; this may cause the table to tip. The table supports up to 10kg of dispersed weight. Do not exceed this limit.**

### 3. Cleaning

Clean the product regularly with a dry and clean cloth.

### 4. Materials

Tube in epoxy painted steel, PP castors and wooden tabletop.

### 5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

### 6. Technical specifications

Check the table at the end of the user's manual. Duration of use : 2 years.

This product has a 12 months warranty, from the date of purchase, against any manufacturing defect.

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