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1994.

**EN-** Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

### 1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, castors...).
- ♥ Check regularly if all parts are tightened.
- ♥ Use only in proper way : don't stand on the table.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ After assembling the table, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ In case you didn't use your table for a long time, let it checked by the technical staff, before using.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### 2. Intended use, Indication and Contraindication

Our over-bed table, is designed to fit over a hospital bed or wheelchair and to provide a patient a stable surface to eat meals upon as well as to be used for a lap-top computer, playing board/card games, reading, or for hobbies.

#### Indications:

Relief of the caregiver when the patient is provided with a nursing bed or a bed frame, as well as predominant bed confinement of the patient; e.g., inability in independent feeding of the patient outside the bed.

#### Contraindications:

- neurological diseases such as tetraplegia
- mental illnesses such as dementia

### 3. Description and Fitting instructions

Our bed table has a tilting foot and top for a functional and comfortable use in all situations, seating or lying down. The whole frame can be folded flat for storage. The laminated top is edged with an anti-shock border allowing books to be held. Supplied with 4 castors, two of them are lockable. Side plates available for all models as an option.

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient.

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier or your authorized Herdegen distributor.
2. The bed table is delivered folded. To unfold it, pull out the locking pin located on the base and move the bed table upwards into a vertical position; release the pin to lock it. Make sure that the pin is fully engaged through the hole, see

- (1).
3. To adjust the height of the plate, unscrew the knurled screw and lift the plate until the desired position is reached, then tighten the knurled screw by screwing clockwise, see (2). Beware, when unscrewed, hold the plate so that it doesn't fall.
4. To recline the plate, pull out the locking pin located under the plate and recline the plate at the desired position, then release the pin to lock it. Make sure that the pin is fully engaged through the hole, see (3).
5. To lock the brakes of the castors, see (4).
6. To add a side plate, recline the plate vertically and insert the side plate into the pipe, then lock it with the allen key and screw it (included), see (5). The side plate remains horizontal.

**Beware : Do not lean on the suspended end of the table top; this may cause the table to tip. The table supports up to 15kg of dispersed weight. Do not exceed this limit.**

**Rental:** This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

- 1- Visual inspection: ensure that the product has not been damaged and check for any defect.
  - 2- Functional inspection: ensure that the product functions properly and repair it if needed.
  - 3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories). Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product.
- Use only appropriate tools to perform the maintenance, cleaning and disinfection.  
 The user's manual is an integral part of the product and must be handed over to every new user.

### 3. Cleaning

Clean the product regularly with a dry and clean cloth.

### 4. Materials

Tube in epoxy painted steel, PP castors and wooden tabletop.

### 5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

### 6. Technical specifications

Check the table at the top of the user's manual. This product has a 24-month warranty, from the date of purchase, against any manufacturing defect. Duration of use: 2 years.

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