# Invacare® Omega

H450 / H450LA

en	Stationary Toilet Chair User Manual .....	4
fr	Chaise-toilettes posée fixe Manuel d'utilisation .....	6
it	Sedia da toilette fissa Manuale d'uso .....	10
es	Silla de inodoro fija Manual del usuario .....	13
pt	Cadeira sanitária estacionária Manual de utilização .....	16
cs	Pevná židle na toaletu Návod k obsluze .....	20
hr	Fiksni toaletni stolac Upute za uporabu .....	23
et	Statsionaarne tualetitool Kasutusjuhend .....	26
lt	Stacionari tualetu kėdė Naudojimosi instrukcija .....	29

This manual **MUST** be given to the user of the product.  
BEFORE using this product, this manual **MUST** be read and saved for future reference.

**Yes, you can.®**

?.1 Images

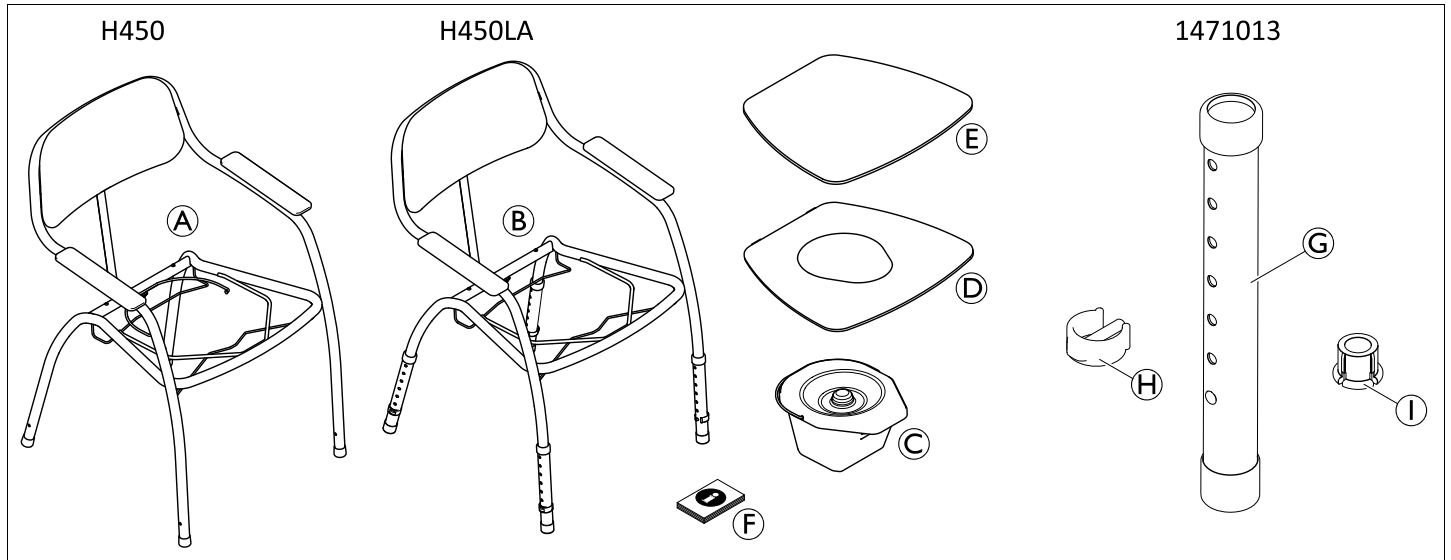
Fig. 1

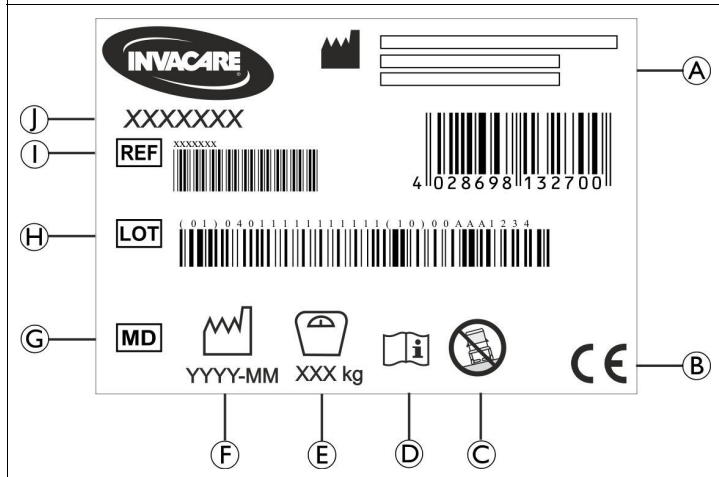
Fig. 2

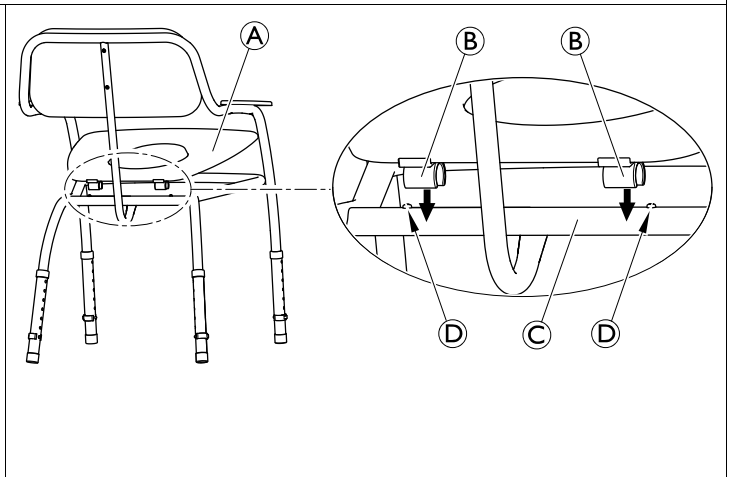
Fig. 3

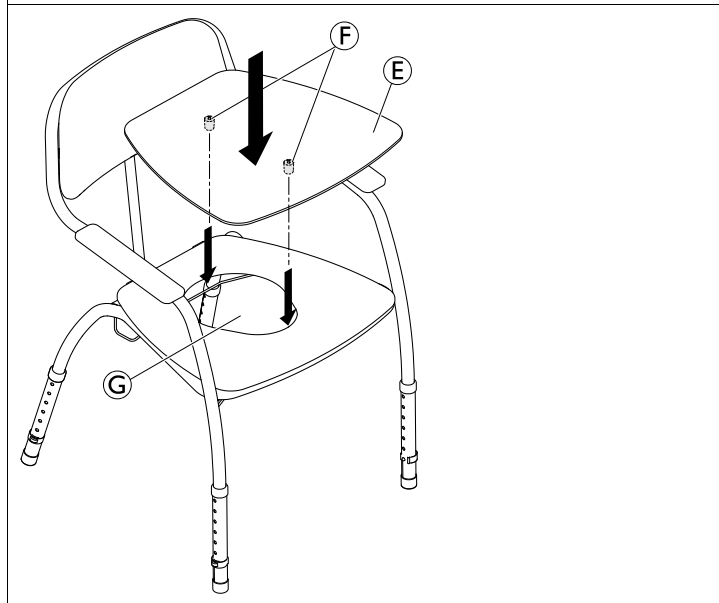
Fig. 4

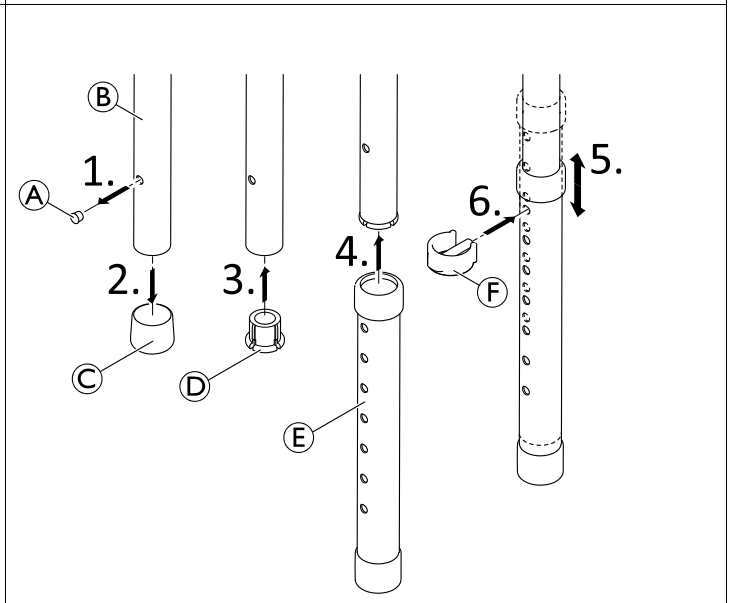
Fig. 5

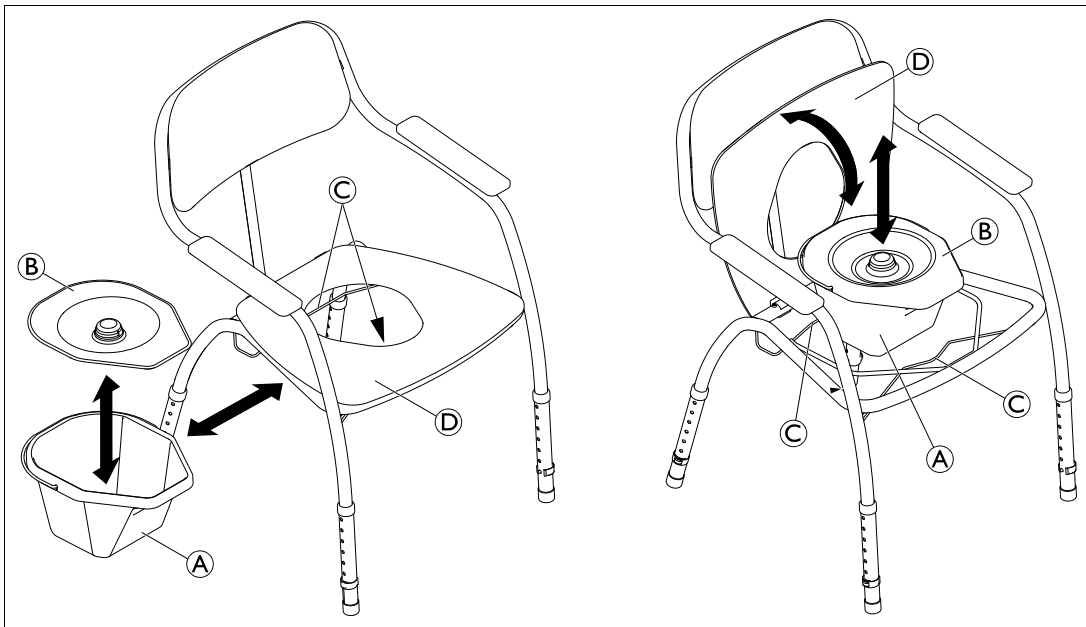
Fig. 6

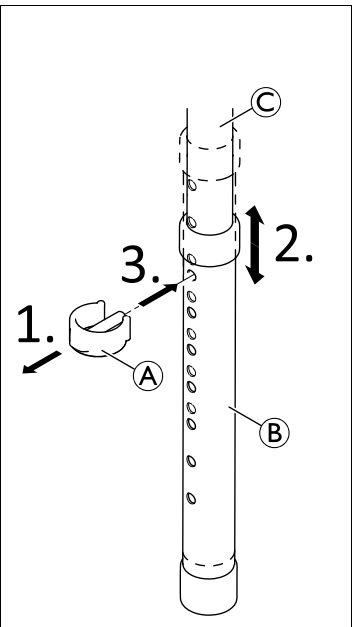
Fig. 7

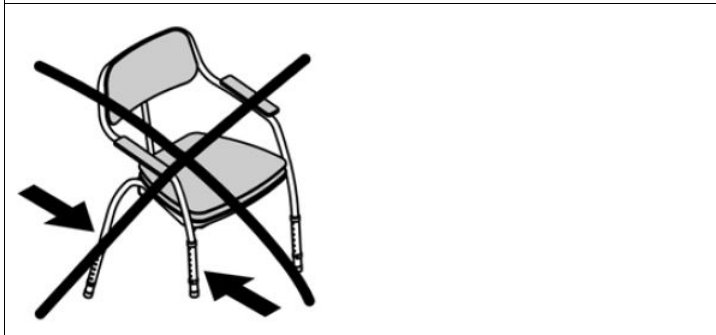
Fig. 8

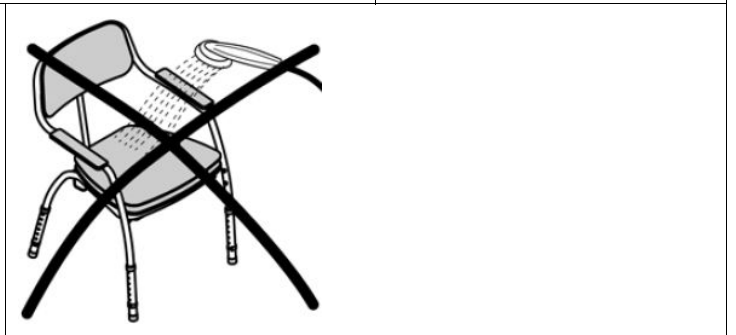
Fig. 9

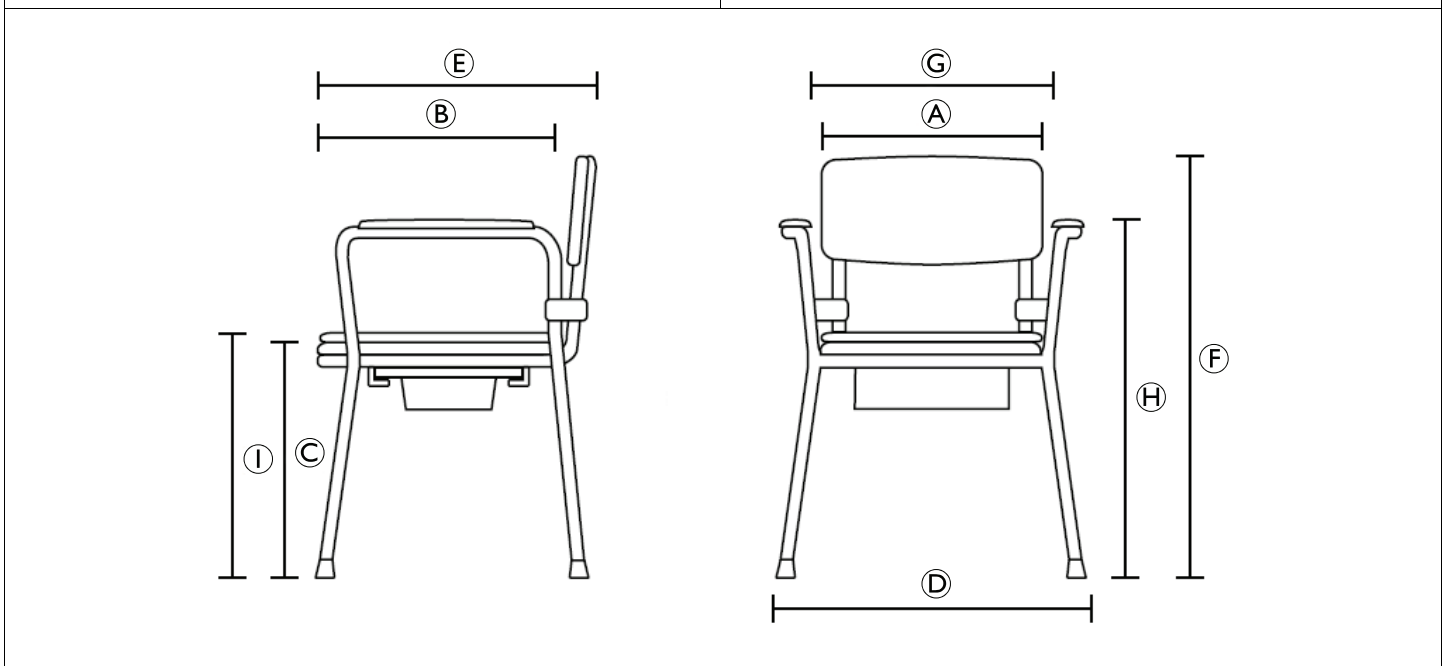
Fig. 10

## 12.2 Cleaning and Disinfection

### General Safety Information

#### CAUTION!

##### Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

#### IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

### Cleaning Intervals

#### IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

### Cleaning Instructions

1. Remove the seat plate from the toilet chair.
2. Clean the toilet chair by hand with water and a mild, non-abrasive cleaner.

The maximum cleaning temperature is 60 °C.

### Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

## 13 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

## 14 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *12 Maintenance, page 5*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 15 Technical Data (Fig. 10)

### H450:

(A)	495 mm	(F)	890 mm
(B)	455 mm	(G)	510 mm
(C)	500 mm	(H)	690 mm
(D)	635 mm	(I)	525 mm
(E)	630 mm		

Product weight	8.3 kg
Maximum user weight	130 kg

### H450LA:

(A)	495 mm	(F)	900 mm – 1050 mm
(B)	455 mm	(G)	510 mm
(C)	510 mm – 660 mm	(H)	700 mm – 850 mm
(D)	635 mm	(I)	535 mm – 685 mm
(E)	630 mm		

Product weight	8.8 kg
Maximum user weight	130 kg

## 16 Materials

Frame	ST37, powder coated
Upholstery (Seat / backrest / armrest pads)	PVC
Toilet pan	PP

All product components are corrosion resistant and do not contain natural rubber latex.

## 17 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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